



PROTEX GLOVES – 1° CLASS MEDICAL DEVICES

USE: Fabric gloves for the prevention and the control of communicable diseases with contact of surfaces.

INSTRUCTION FOR USE: wear gloves with clean and dry hands.

PRECAUTIONS:

Do not use with damaged skin.

Be careful how remove the glove to avoid contact with the outer surface that may have come into contact with microorganisms.

In order to ensure the effectiveness of the treatment, check that the mesh is intact and that the gloves do not show any tears and/or cuts.

Use a hand cream after removing the gloves. Before reusing let dry the inside of the gloves.

WASHES:

Gentle wash cycle, hand wash with mild detergent, without fabric softener.

Reactive the water-repellent treatment with moderate heat after every wash, dry glove with iron maximum 110 C, no steam and without leaning or delicate dryer max 80C. An average of 10 washes.

STORAGE CONDITIONS:

The medical device is no longer able to perform its functions after 10 washes and/or after the expiry date indicated in the package, even if the product is packaged.

Storage should be done in a cool and dry place, avoiding excessive heat.

WARNINGS:

For any problem notifies the manufacturer, communicating the batch number on the label and the problems emerged. The manufacturer accepts no responsibility for improper use of the product.

FIELD OF APPLICATIONS:

Medical sector in contact with screen and touch devices; pharmaceutical sector; transport, companies, logistics, sports, commercial, catering.

STORAGE TRANSPORT AND DISPOSAL:

Store in a clean and dry area. Handle with clean and/or sanitized hands. Transport inside their specific appropriate packaging bags. The medical device can be disposed of in undifferentiated waste.

TECHNICAL SPECIFICATIONS:

8341F00 MODEL

Glove composition: Q-SKIN yarn enriched with silver ions with a bacteriostatic and anti-odour action (ISO 20743 - ATCC 6538P)
95% Polyamide (Q-SKIN®)
4% Elastomer, 1%AG

APPLICATIONS IDEALS FOR:

- Indoor use to guarantee protection
- Storage operations
- Sanitary use
- Light assembly operations
- General manipulations
- Jewellery store
- Medical environments with electronic devices
- Field of sport
- Transports



Registering DM CLAS I, NR 1981306 – Health Ministry

MADE IN ITALY



ITALIAN CONVERTER Srl
Str. Fogliano S. Marco, 76/78
27029 Vigevano (PV)

